

Help Page - Android

[Rapid Rhythm Reader App](#)

[KardiaMobile®](#)

[Samsung Galaxy™ Watch](#)

****Rapid Rhythm Reader does not check for heart attacks.**

****Rapid Rhythm Reader does not replace ordinary medical care.**

****The purpose of Rapid Rhythm Reader is to evaluate heart rhythm recordings obtained from mobile devices in persons without feelings of chest, arm, or jaw discomfort, shortness of breath, severe weakness, uncomfortable heart pounding, dizziness, or passing out. *If you have any of these feelings, you should seek medical attention at once.***

Where are my interpretations?

On the home screen of the *Rapid Rhythm Reader* app appears “My ECGs.” This is where a list of your ECG(s) that have been submitted will appear. An hourglass symbol indicates the recording has been submitted, but not yet reviewed. A red heart symbol indicates the ECG rhythm has been reviewed by the Cardiologist and is ready for your viewing.

I cannot see my interpreted ECGs:

1. On the home screen entitled “My ECGs,” pull down the page until a refresh wheel appears.
2. A red heart symbol indicates an interpretation is ready for your viewing.
3. If an interpretation has not appeared within 24 hours of recording submission, email us at help@healthalert.io

How to send KardiaMobile® recordings for review:

The KardiaMobile device is used to obtain an ECG onto a compatible phone with the Kardia® app installed.

1. Open the Kardia app on your Android phone and select the “RECORD YOUR ECG” tab and follow the instructions on the next screen(s).
2. Tap on the ECG you see.
3. Click on the “SHARE” (an envelope icon).
4. Select “Email PDF.”
5. Do not password protect your PDF (select “SKIP”).

6. When the share screen comes up, select *Rapid Rhythm Reader* from your list of apps.
7. Confirm in the dialogs that appear.

You can also send an ECG that has been previously recorded.

1. Select “History” from the menu at the bottom of the screen.
2. Select an ECG that you want to send using “SEE EKG HISTORY” tab.
3. Proceed as from step 3 above.
4. Another method is to scroll down in the home screen until you see the “Health Diary.” The forward and back arrows can be used to select a date on which an ECG was recorded. Select the ECG when you see it and proceed with sharing.

How to send *Samsung Galaxy™ Watch* recordings for review:

Although an appropriate *Samsung Watch* is used to obtain an ECG, a *Samsung Phone* is used to transmit an ECG recording for review.

You must have installed the *Samsung Health Monitor* app on both your *Samsung Watch* and *Samsung Phone* to be able to record an ECG.

1. Select the ECG app on the Watch and follow all instructions. (Pay particular attention to all health precautions provided by Samsung. Use of *Rapid Rhythm Reader* does not remove any restrictions on appropriate usage of the *Samsung Health Monitor* app.)
2. Open *Samsung Health Monitor* on your *Samsung Phone*. You will see a graphic of the ECG just recorded.
3. Tap on the “Share this report” bar and select *Rapid Rhythm Reader* from the list of apps presented.
4. Confirm in the dialogs that appear.

You can also send an ECG that has been previously recorded.

1. Open the *Samsung Health Monitor* app and select the “View history” tab.
2. Select the ECG you want to send us by looking into history and selecting a tracing.
3. Tap on the “Share this report” bar and select *Rapid Rhythm Reader* from the list of apps presented.
4. Confirm in the dialogs that appear.