

Help Page - iOS

[Rapid Rhythm Reader App](#)

[Apple Watch®](#)

[KardiaMobile® on iPhone®](#)

****Rapid Rhythm Reader does not check for heart attacks.**

****Rapid Rhythm Reader does not replace ordinary medical care.**

****The purpose of Rapid Rhythm Reader is to evaluate heart rhythm recordings obtained from mobile devices in persons without feelings of chest, arm, or jaw discomfort, shortness of breath, severe weakness, uncomfortable heart pounding, dizziness, or passing out. *If you have any of these feelings, you should seek medical attention at once.***

Where are my interpretations?

On the home screen of the *Rapid Rhythm Reader* app appears “My ECGs.” This is where a list of your ECG(s) that have been submitted will appear. An hourglass symbol indicates the recording has been submitted, but not yet reviewed. A red heart symbol indicates the ECG rhythm has been reviewed by the Cardiologist and is ready for your viewing.

I cannot see my interpreted ECGs:

1. On the home screen entitled “My ECGs,” pull down the page until a refresh wheel appears.
2. A red heart symbol indicates an interpretation is ready for your viewing.
3. If an interpretation has not appeared within 24 hours of recording submission, email us at help@healthalert.io

How to send *Apple Watch* recordings for review:

Although the *Apple Watch* is used to record an ECG, the *Health* app on the paired *iPhone* is used to send an ECG for review:

1. Record an ECG using your *Apple Watch*.
2. On the paired *iPhone*, when your ECG appears, tap the share icon or select “Export a PDF...” and then tap the share icon.
3. Select *Rapid Rhythm Reader*.
4. If you agree, confirm by selecting “OK” in the dialog box that appears.

You may also send an ECG that you recorded previously:

1. On the paired *iPhone*, tap “Browse” on the lower navigation menu of the Apple *Health* app.
2. Tap “Heart.”
3. Select an ECG within the ECG section
4. Tap the share icon.
5. Select *Rapid Rhythm Reader*.
6. If you agree, confirm by selecting “OK” in the dialog box that appears.

How to send *KardiaMobile* on *iPhone* recordings for review:

Although the *KardiaMobile* is used to record an ECG, the *iPhone*® is used to send an ECG for review:

1. Follow instructions in *KardiaMobile* Quick Start Guide.
2. In the *Kardia*® app on the *iPhone*, scroll down to the “more options” section and select download PDF.
3. On Password Protect PDF pop-up screen press “Skip.”
4. Tap the share button in top right corner, and select the *Rapid Rhythm Reader* app.
5. Select “OK” on the sharing ECG pop-up screen.
6. Select “OK” on the next pop-up screen.

In order to send us an ECG that you previously recorded:

1. Tap “History” at the lower navigation menu of the *Kardia* app on the *iPhone*.
2. Tap “EKG,” or “See EKG History.”
3. Tap mail icon of the ECG you wish to send.
4. On Password Protect PDF pop-up screen press “Skip.”
5. Tap the share button in top right corner, and select the *Rapid Rhythm Reader* app.
6. If you agree, confirm by selecting “OK” in the dialog box that appears.